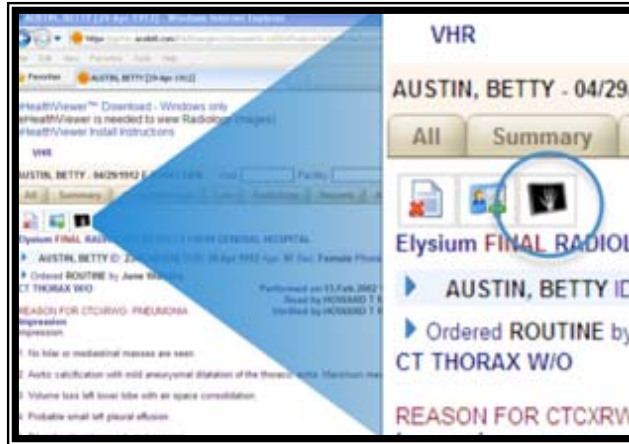


The Rochester RHIO gives you the complete picture for every patient.



1. Open a Radiology or Cardiology report as you normally would using the VHR.
2. At the top of the report an “X-Ray Hand” icon will appear for studies that have images available.
3. By selecting this icon, you will automatically launch the eHealthViewer® application, and the associated images will be automatically loaded for viewing.
4. Recent exams are held in storage for rapid loading. Older exams are also available, but may need to be retrieved from their original PACS location, which will require additional time to load. .
5. Before using this new functionality for the first time, make sure you download and install the eHealthViewer application.

Installing the eHealthViewer

On the first page users see once they log on to the VHR, choose to download the eHealthViewer application..





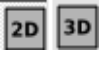

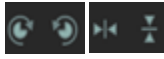

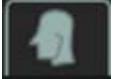

1. When prompted, click “Run”. Do NOT save the file.
2. You may see a security warning, click RUN again.
3. Then click Next>Next>Fullscreen>Everyone>Next>Close.

Installation is complete!

For support, please call: 877-865-RHIO(7446)



**BETTER INFORMATION
FOR BETTER PATIENT CARE**

MOST USED FUNCTIONS	
 <p>Undo: Ctrl + Z Redo: Ctrl + Y</p>	<p>Home, Undo, and Redo: You always have the ability to return to the previous screen or your starting screen layout.</p>
 <p>Pan: Ctrl + LT click Zoom in: Shift + LT click Zoom out: Shift + RT click</p>	<p>Pan, Zoom, and Cancel Pan/Zoom: Zoom by choosing the tool & click on the image. Pan by selecting the tool & dragging the image.</p>
 <p>Middle + click & drag</p>	<p>Width/Level: Select the tool & grab the yellow sections of the crosshairs in order to adjust window width & level.</p>
	<p>Display Set Layout: Change screen layout i.e. change from 2x1 to 2x2 window panes.</p>
	<p>CT & MR images can also be viewed in 3D by selecting the 3D CAM icon. Return to 2D viewing by selecting the 2D CAM icon.</p>
	<p>Measurement Tools: 2 point distance, Distance, 3-point angel, 4-point angle, Circular intensity, & Freehand intensity. Select the tool & click on the image to set the measurement points.</p>
	<p>Rotate Clockwise & Counter Clockwise. Mirror Horizontal, Mirror Vertical.</p>
<p>Paging</p> 	<p>Page through series of images. Icon appears when you place the mouse over the center of the image. Click & hold left mouse while moving up & down.</p>
<p>Sidebar—Best way to select new studies.</p>	<p>Move cursor to left side of window where you will see a line of small dots > Sidebar will pop up.</p>
<p>Patient Study</p> 	<p>See all the patient exams & series. Click on (+) to expand the list and (-) to contract. "Drag & drop" with the mouse any series to the image window to load into the display.</p>
<p>Rotation Box Handle</p> 	<p>Grab the top left corner of the box surrounding the image, & a rotation box handle will appear. Rotate the image to see different views.</p>
<p>Other Functions</p>	<p>Double click on any image to view it in full screen mode.</p>
<p><i>For further assistance, please call (877)865-7446 or contact info@qrrhio.org.</i></p>	